



🌐 [heygears.com](https://www.heygears.com) ✉ info@heygears.com
☎ Tel: +1 (318) 353-4295 (Global)/ +1 (949) 418-9418 (USA) / +44 148-396-8549 (EU)
📍 USA-17931 Sky Park Circle, Suite E, Irvine, CA, 92614
CHN-Block B2, 501, 601, Enterprise Accelerator, Kaifa District, Guangzhou City

Follow us @HeyGears




Quickstart V1.3



AirCure

ChairSide Curing Machine

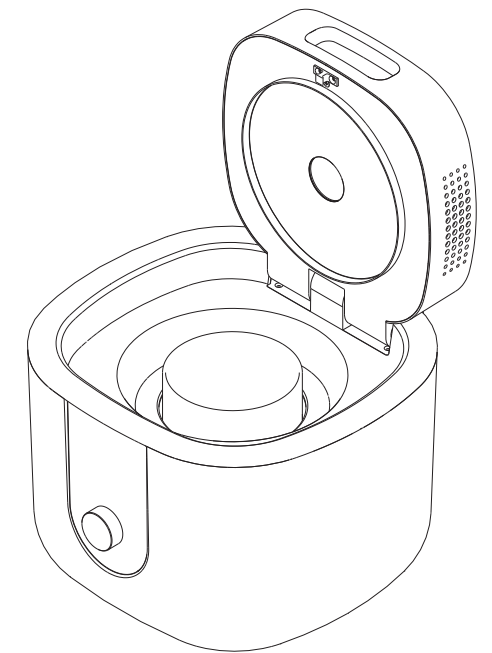

Please read this installation and operation manual carefully before use

Thank you for choosing HeyGears. **For your safety, please follow the operating instructions.** If you fail to comply with the instructions in this manual, HeyGears won't be responsible for the consequential losses unless required by law.

Please keep this operation manual properly for future reference. The pictures are for reference only and the products are subject to the actual product.

1 Items & Parameters



AirCure



Power Cord




Beaker



Brush

Name	UltraCraft Post Curing Unit	Input	100–240V~ @50/60Hz
Model	UltraCraft AirCure	USB Output	5 V == 500 mA
Size	310 x 300 x 245 mm	Power	123 W
Weight	6 kg		

2 Overview

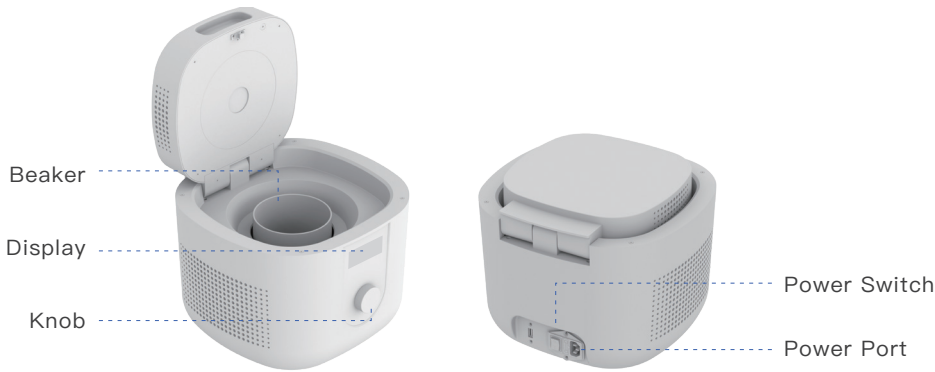

3 Installation Requirements



Place the product horizontally on a stable table before use.



Make sure there is a 100V–240V power outlet nearby.



Don't open the lid during curing which will lead to stop.



Beaker needs to be cleaned with brush regularly after finishing the curing.

4 Operation Steps

STEP 1

Plug the supplied power cord into the printer power port and turn on the switch next to it.




STEP 2

1. Remove the model from the build platform, cut the support with nipper, then put water and model into the beaker.




2. Models printed with Denture 2.0, Denture Teeth, Temp C&B 2.0 have to be cured with G reagent. Steps: Add G reagent to the beaker–Put model in and close the lid–Start curing with AirCure.

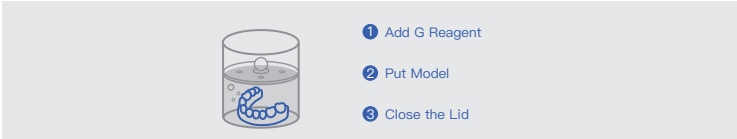

STEP 3

Place the beaker with the model into the AirCure, close the lid and select the material by rotating knob, then press to start.
*You can long press the knob for canceling.

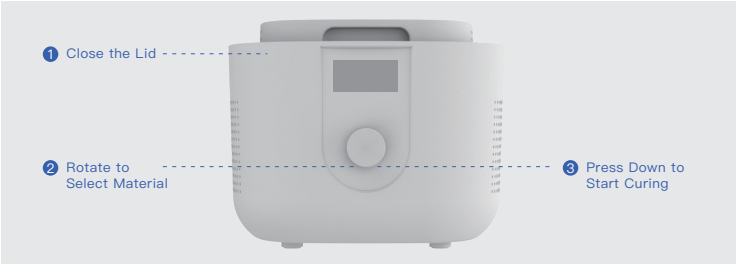

STEP 4

Remove the models when finished.



5 Maintenance and Technical Support

You can contact the after-sales and technical support staff via the following ways:
1. After-sales support email address: support@heygears.com
2. Available worldwide: +1 318–353–4295
3. Europe: +44 148–396–8549
4. USA: +1 949–418–9418
5. Visit <https://www.heygears.com/services>